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**ORTHODONTIC FACEMASK** **CE**  
INSTRUCTIONS FOR USE

**GENERAL INFORMATION**

Orthodontic Facemask is used as a treatment option for Class III malocclusions. After being adapted and shaped to the dimensions of the patient's face, the mask is placed on the forehead and on the chin by means of the forehead and chin support and connected to various devices already in place in the patient's mouth by means of elastic bands that are hooked to the grill supports. In this way, anterior-posterior traction forces are obtained which allow the altering of the position of the teeth and the maxilla.

The pack contains:

- 1 orthodontic facemask
- 1 hex key for adjusting

The device is available in two sizes, MINI and MAXI, and in different colours. Information on the purchased device is shown on the label affixed to the packaging.

The Orthodontic Facemask is a multipurpose single-patient medical device and can be used on patients from six years of age.

For US market only can be used on Adolescents in the age group of 12 <18 years old.

The device is subject to medical prescription. It is the responsibility of the dentist to evaluate the most suitable treatment based on the patient's clinical conditions.

These instructions for use also contain information intended for the patient. The dental practitioner is responsible for providing the relevant information. A copy of the instructions for use can also be downloaded from the Manufacturer's website.

**CAUTION (only for US market)**

Caution: Federal law restricts this device to sale by or on the order of a dentist.

**WARNING (only for California market)**

This product can expose you to chemicals including nickel and chromium, which are known to the State of California to cause cancer, and chromium, which is known to the State of California to cause birth and other reproductive harm. For more information go to www.P65Warnings.ca.gov

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**CONTRAINDICATIONS**

The device contains nickel and chromium. Patients allergic to these metals should not be treated with the device.

**SAFETY WARNINGS**

It is the responsibility of the dentist to inform the patient of the contents of these instructions for use or to ensure they are provided.

Use the device exclusively in accordance with the instructions given by the dentist in charge of treatment and only use any accessories and elastic bands supplied.

The device must only be used on healthy skin; in the event of redness of the parts in contact with the device, temporarily suspend its use.

The device must not be used during sporting activities, complex manual work and any dangerous situations resulting from moving objects in general. Do not derive while wearing the device.

The dental practitioner is responsible for verifying the suitability of the device. If the device is damaged, do not use it and contact the manufacturer.

In case of breakage, damage or other fault, do not use the device and contact your dentist.

If the PVC Coating pads in contact with the skin show wear or deterioration, contact your dentist.

In the event of a serious accident occurring in connection with the device, immediately inform the manufacturer and the competent authority of the area in which the user and/or the patient is based.

The device must be put on with the help of an adult.

**SETTING UP**

The device is a product for orthodontic therapies that can only be used and set up by qualified personnel (dentist).

Indications on how to set up the device correctly:

- Place the chin rest (A) on the patient's chin, making sure that it does not press on the lower teeth.
- Move the forehead rest (C), positioning it over the patient's eyebrows, secure it in the desired position with the fixing screws on the sides by means of the hex key supplied.
- Adjust the height of the grill (E) with the adjustment screws (D), located on the back, using the hex key supplied.
- Connect the elastic bands to the brackets or wire hooks and pull them forward to hook them to the corresponding supports (F) positioned on the grill (E).

**Do not force the tightening of the screws to prevent thread loss.**

The dentist is responsible for using suitable brackets, elastics or wire hooks. These devices must bear the CE marking.

Once being used on the patient, the device requires periodic checks carried out by the dentist who will evaluate the safety of the device installed on the patient according to the tensile force of the elastic bands applied.

**STORAGE, CLEANING AND MAINTENANCE**

When not in use, place the device in a dry place, away from heat sources and protect it from accidental impacts. It is recommended to store it in a suitable container or case.

**Do not stack objects on the device.**

Before each use, clean the parts in contact with the skin using a cloth moistened with water and pH neutral soap.

**DISPOSAL**

At the end of the useful life, dispose of the non-disassembled device according to the regulations in force in the respective area.

KEY	
SYMBOL	DESCRIPTION
	Manufacturer
	Item code
	Batch number
	Date of manufacture
	Medical device
	Unique Device Identifier
	Please read the instructions for use
	Read the warnings
	Device compliant with MDR (EU) 2017/745
	Caution: Federal law restricts this device to sale by or on the order of a dentist
	Keep away from light
	Keep dry

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